

Aladin₂ Cassettes

Anesthetic agent vaporizer with electronic control

Electronic control – a unique technology for vaporizing anesthetic agent

The electronic control on the Aisys™ CS² Anesthesia Delivery System provides unique benefits such as automatic record keeping of set agent concentrations and agent usage calculations. Agent-specific Aladin₂™ Cassettes can be used to deliver isoflurane, sevoflurane and desflurane anesthetic agents.



Our enhanced design provides:

- Integrated overfill protection
- On screen, electronic level sensing and agent identification
- Five filling options:
 - Easy-Fil™ Filler System
 - Safe-T-Seal™ (SEV) Integrated Valve Filler System
 - Quik-Fil (SEV) Filler System
 - Piramal Fill (SEV) Filler System
 - Safe-Fil™ (DES) Filler System
- A prismatic site glass for clear indication of liquid agent level
- Electronic Control allows for automatic record keeping, gas usage calculation, fresh gas flow compensation, and temperature/pressure compensation
- Each cassette is magnetically coded for its specific agent allowing the Aisys CS² anesthesia machine to automatically identify the anesthetic cassette being used
- Lightweight cassette allows for easier handling
- No planned factory service required
- Vaporizer self-check and diagnostics



Agent measurements

Electronic agent delivery control and level sensing

How it works

In the Aisys CS² system, anesthetic agent control consists of two parts: the electronic control mechanism in the anesthesia machine, and the separate Aladin₂ agent cassette.

Agent concentration is adjusted by regulating the amount of fresh gas flowing through the cassette. A proportional valve is used to regulate the flow. Part of the fresh gas bypasses the cassette, so the more fresh gas allowed to pass through the cassette, the greater the concentration. Fresh gas flow along with flows into and out of the cassette are electronically measured for safety.

Each Aladin₂ cassette is magnetically coded. This allows the Aisys CS² anesthesia system to recognize which type of agent cassette is inserted.

The agent vaporizes freely in the cassette without pressurization or heating. Pressure, temperature and fresh gas content (O₂ and N₂O/Air) are taken into account to help achieve greater accuracy when compared to traditional vaporizers.

Recommended filling procedure is to remove the cassette from the system before filling, and place the cassette on a flat surface while filling.

Benefits

Convenient

- The Aladin₂ cassette can be handled or stored in any position
- Weighing less than 3.4 kg/7.5 lbs, the Aladin₂ cassette can be easily carried and handled

Automatic record keeping and gas usage calculation

- Electronic control of desired agent concentration provides agent setting data for automatic record keeping
- Gas usage data provides a unique tool for low-flow savings analysis

Enhanced safety

- During the recommended daily system check, the Aisys CS² anesthesia system performs a check on the Aladin₂ cassette and electronic vaporizer
- Aisys CS² agent control is monitored every 200 milliseconds during operation

Easy-Fil bottle adapters



Physical specifications

Dimensions

Height	7 cm/2.76 in
Depth	24 cm/9.45 in
Width	14 cm/5.51 in
Empty weight	2.8 kg/6.2 lb

Cassette handling

No restriction for tilting during storage or handling.

Agent capacity	Iso, Sev	Des
Total	220 mL	Des 240 mL
Normal fill when indicator shows empty	125 mL (95 mL residual volume)	140 mL (100 mL residual volume)


Filling

Overfilling protection	Overfilling prevention system built into the cassettes
Filling system	Easy-Fil: Filler system for Isoflurane and Sevoflurane Quik-Fil: Filler system compatible with AbbVie’s system for Sevoflurane Safe-T-Seal: Filler system compatible with Baxter’s system for Sevoflurane Safe-Fil: Filler system compatible with Baxter’s system for Desflurane Piramal Fill: Filler system compatible with Piramal’s system for Sevoflurane
Filling speed	> 2 mL/s

Environmental specifications

Temperature

Operating range	10° to 35°C/50° to 95°F
Storage range	-25° to 50°C/-13° to 122°F

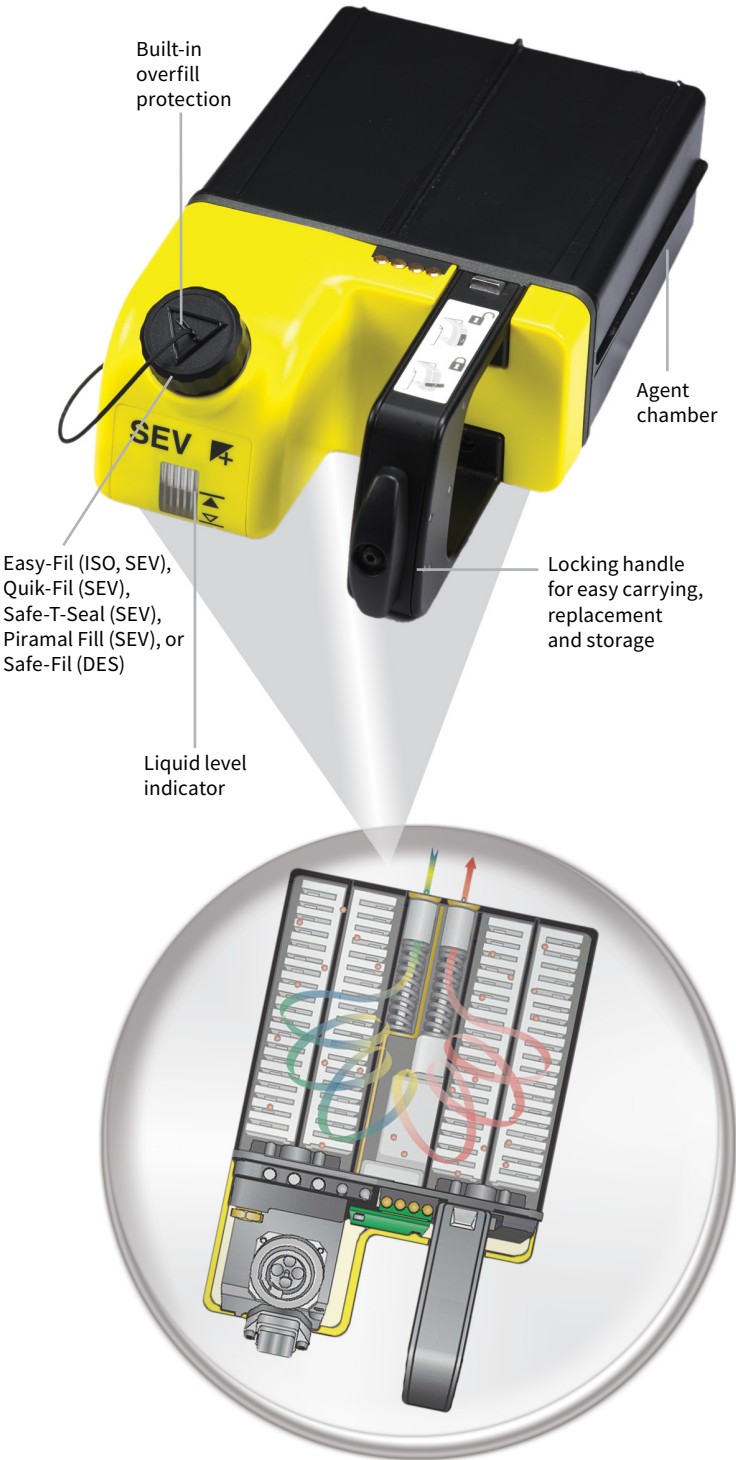

Electronic specifications

Agent setting ranges

Isoflurane	OFF, 0.2 to 5% in fresh gas flow, resolution 0.1%
Sevoflurane	OFF, 0.2 to 8% in fresh gas flow, resolution 0.1%
Desflurane	OFF, 1 to 18% in fresh gas flow, resolution 0.2%



Accuracy

Isoflurane and Sevoflurane

In typical operating conditions:	
Ambient temperature	18° to 25°C ±10% of setting or ±0.2% v/v (whichever is greater)
Fresh gas flow range	1 to 10 L/min

In other operating conditions:	
Ambient temperature	10° to 35°C ±20% of setting or ±0.4% v/v (whichever is greater)
Fresh gas flow range	0.2 to 15 L/min

Desflurane

In typical operating conditions:	
Ambient temperature	18° to 25°C ±10% of setting or ±0.5% v/v (whichever is greater)
Fresh gas flow range	1 to 10 L/min

In other operating conditions:	
Ambient temperature	10° to 35°C ±20% of setting or ±1% v/v (whichever is greater)
Fresh gas flow range	0.2 to 15 L/min

The filler ports depicted in this datasheet may not be available in all countries and regions. Contact your GE Healthcare representative to see if a specific filler port is available in your country. The Aladin₂ Cassette for Desflurane with Piramal Fixed Filler (not shown) is not cleared in the USA. Not available for sale in the United States.

Products mentioned in the material may be subject to government regulations and may not be available in all countries. Shipment and effective sale can only occur after approval from the regulator. Please check with local GE Healthcare representative for details. Aisys, Easy-Fil and Aladin₂ are trademarks of GE Healthcare. GE and the GE monogram are trademarks of General Electric Company. Used under trademark license.

©2023 GE Healthcare.

Baxter, Safe-T-Seal and Safe-Fil are trademarks of Baxter International Inc.

All other product names are trademarks or registered trademarks of their respective companies.

DOC2757540 01/2023



GE Healthcare